

Titan® E-lectric Integrated Control Unit Mounting Instructions



Tools Required:

7/16" wrench, 3/8" wrench, 17/64" drill bit, power drill, and pencil

Titan E-lectric Unit Mounting, StarDental Part# 263754

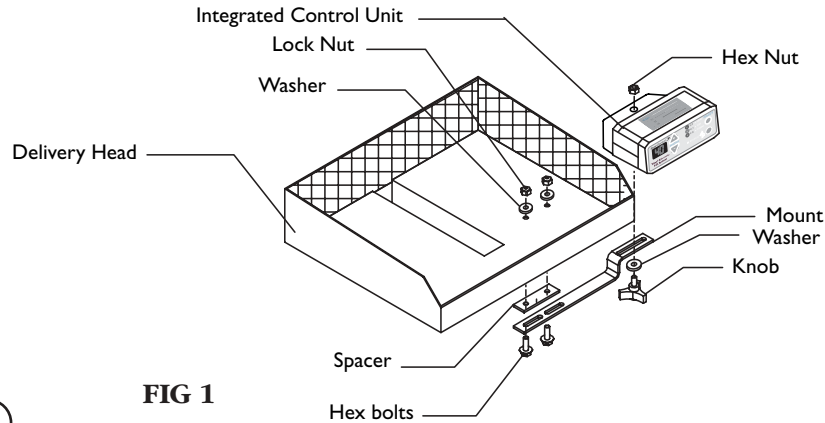


FIG 1

1

1. Determine a suitable mounting location: front, back, left, or right.

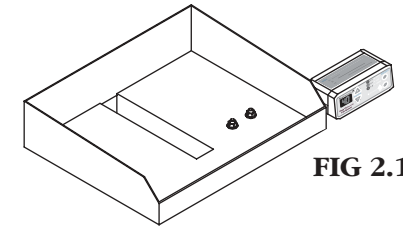


FIG 2.1

2. Use the suggested mounting position or manually determine location.

3. To use the suggested mounting location, see FIG 2.3. You may mount on left or right.

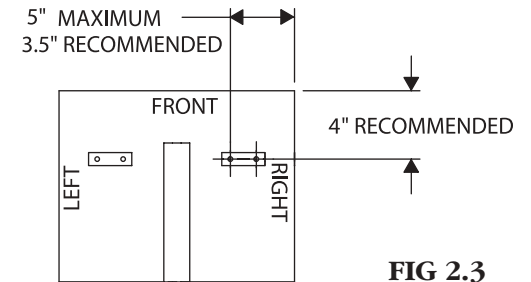


FIG 2.3

4. To manually determine a mounting location, assemble mount to control unit using knob, see FIG 3.5. Allow knob to remain loose so control unit is free to rotate.

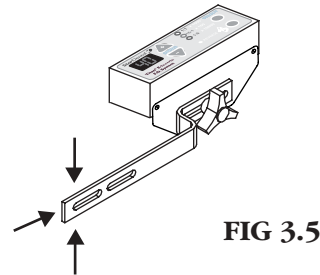


FIG 3.5

5. Position control unit in desired location. Press mount flush against mounting surface. With a pencil, mark underside of mounting surface at three edges, see FIG 3.5.

6. Loosen knob and remove control unit.

7. Align spacer with marks made from step 5. Ensure hole location won't interfere with internal components at mounting position. Use the 17/64" bit to drill two holes through mounting surface at spacer hole location.

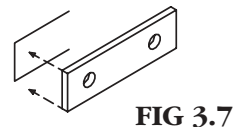


FIG 3.7

3

8. Place bolts in mount and spacer, then through bottom side of mounting surface, see FIG 4.8.

9. Secure mount using two washers and lock nuts, see FIG 1.

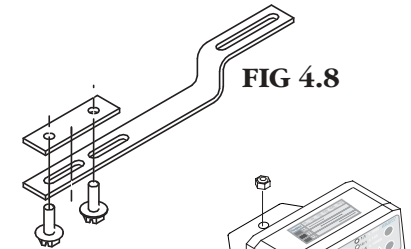


FIG 4.8

10. Secure control unit to mount with knob, see FIG 4.10.

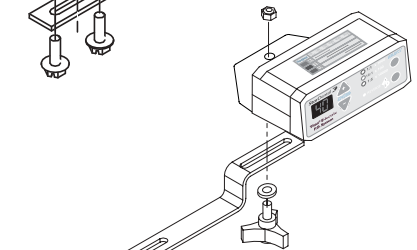


FIG 4.10

4