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Contains Annatto and Jua / Contiene Urucú y Juá

Cleaning.

There are no sterilizing requirements for bottling the products of the kit. However, we keep appropriate handling and cleaning conditions. This procedure is carried out without contact with the environment and under cares that range from the use of disposable gloves to the sterilization of the tools used for bottling by immersing the parts in an appropriate disinfection solution. There are no sterilization requirements for the product itself because it is not ingestible or injectable.

Emissions.

There is the production of potassium and fluorine ions and oxygen with the purpose described in the treatment. There is no thermal or light emission, ionizing radiation or volatile gas emission that can endanger the health of the patient or the professional, as long as the appropriate safety equipment is used (protection goggles).

Side Effects

Hypersensitivity cases after treatment are reported in literature. In such cases, use the desensitizer that comes with the kit. In more severe cases, the use of analgesic is recommended to provide more patient comfort.

GENERAL SAFETY INSTRUCTIONS:

The product should be handled by capable professionals, equipped with appropriate protection goggles and gloves if any photoactivation device is used.

The patient should also use protection goggles and all necessary precautions should be taken in order to avoid the accidental contact of the product with skin, mucosa or clothes.

The high oxidation power of Lase Peroxide Sensy⁽¹⁾ makes any contact with live tissues cause a temporary rash, with some whitish spots that normally disappear within a 2-hour period. If there is an accidental contact with skin or intra-oral mucosa, the area should be cleaned and washed with plenty of water. In these cases, the action of peroxide can be neutralized by covering the area affected with some drops of 10% sodium bicarbonate (supplied in the kit).

If there is an accidental contact with the eyes, wash them immediately with plenty of water and contact a specialist.

The Phase 1 bottle – Hydrogen Peroxide (H₂O₂) – can be under pressure and/or present leakage due to adverse storage conditions. Before opening the bottle with Peroxide, protect your hands and eyes and watch for leaked liquids inside the Kit. Small proportion leakages do not compromise the use of the product. In cases of larger leakages, contact DMC Equipamentos. Wash your hands and materials contaminated with peroxide after contact with the product.

ALWAYS BEAR IN MIND THE PREVENTION OF ACCIDENTS
AND SAFETY WHEN HANDLING THE PRODUCT.

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After the previous prophylaxis (3 days beforehand), register the initial coloring of dental elements.

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If relative isolation is chosen, place a lip retractor.

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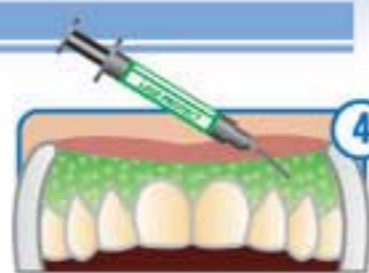
Dry the teeth and the gingival tissue area before applying Lase Protect⁽²⁾. Then remove the syringe cover and attach the application tip.

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
Apply Lase Protect⁽²⁾ in thin layers covering about 0.5 cm of the gingival tissue around the area, sealing the entire region that is going to be treated, including root exposures.

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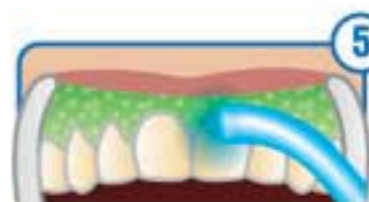Cure Lase Protect⁽²⁾ for at least 30 seconds prior to whitening agent application.

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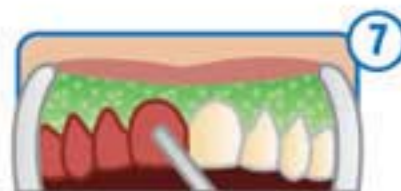
Prepare the gel by adding Phase 1 – Hydrogen Peroxide (H₂O₂) and Phase 2 - Thickener (at the ratio of 3 drops of peroxide for every drop of thickener). Mix

immediately with the help of a spatula. The amount of mixture should be according to the number of teeth and the area, keeping the ratio of 3:1 (one tooth).

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Apply the gel on the vestibular surface of the teeth with the help of a spatula or syringe. A 1mm to 2mm thick gel layer should be applied all over the vestibular enamel.

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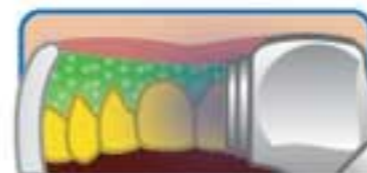
Irradiate with Whitening Lase⁽³⁾ for 3 minutes, let it rest for other 2 minutes, apply light for 3 more minutes

Sistema Ultra Blue IV⁽⁴⁾

and let it rest for 2 minutes. In this stage, the color of the product will have changed from red to yellow.

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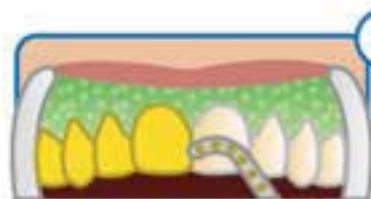
Irradiate one sector with Ultra Blue IV System⁽⁴⁾ for 2 minutes (right side), invert the side (left side) and irradiate for 2 minutes. Then irradiate the central part (incisors) for 2 minutes. After three irradiations of the first step, let the whitening agent rest for 1 minute. Repeat the application sequence.

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
Each gel application should last from 10 to 15 minutes. After this time, remove the gel with the aspirator tip and clean the surface with some gauze.

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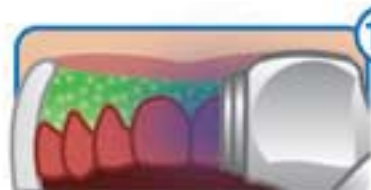Repeat procedures from 6 to 9 once or twice more, depending on the result obtained.

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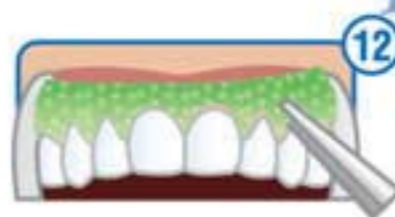
After the end of the third whitening agent application, remove with an aspirator and wash with plenty of water.

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Remove the gingival barrier by pulling it out.

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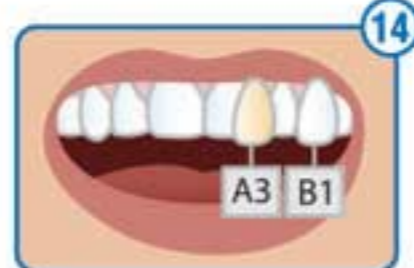
- 13** Polir os elementos dentais utilizando o disco impregnado⁽⁵⁾ umedecido.

Polish dental elements by using a damped impregnated disc⁽⁵⁾.

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Perform a final color registration. Apply desensitizer gel.
Note: If more than one session is necessary, there should be a minimum interval of 7 days between them.

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IMPORTANT

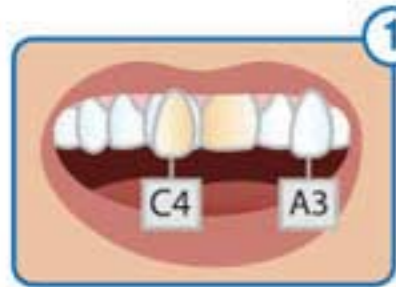
- The light emitter equipment (blue LED (visible) and infrared (invisible)) used should present an energy of at least 250mW/cm² irradiance. LED or LED/Laser systems with low light energy are not indicated for this teeth whitening technique.
- The use of bicarbonated jet immediately before the whitening procedure can inhibit the action of the whitening agent, since the aforementioned composite is the neutralizing agent of the hydrogen peroxide that composes the kit.
- If the whitening gel is being stored under refrigeration, it is advisable to use it only after 30 minutes in room temperature.
- No desensitizer product should be used immediately before the Lase Peroxide Sensy⁽¹⁾ application.

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Perform coronary opening and removal of restoration material and residues. Remove gutta percha from the pulp chamber up to 3mm above the amelo-cemental junction, entering the radicular duct.

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Make peripheral seal with glass ionomer or zinc phosphate cement at least 2mm thick.

4 Aplicar a barreira gengival por vestibular proximais e lingual/palatino com espessura de 0,5 cm protegendo toda área de tecido gengival. Fotopolimerizar a barreira gengival por 30 segundos.

Apply gingival barrier per proximal vestibular and lingual/palatine with a thickness of 0.5 cm, protecting the entire gingival tissue area. Cure gingival barrier for 30 seconds.

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Clean pulp chamber before preparing the gel. 6 drops of Phase 1 with 2 drops of Phase 2 are enough to whiten one pulpless tooth.

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Apply a layer of Lase Peroxide Sensy⁽¹⁾ whitening gel with a thickness of 2mm per vestibular and inside the pulp chamber. The gel should be applied 3 times in the same session and in every application, remain from 10 to 15 minutes in position.

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Irradiate the vestibular surface with Whitening Lase⁽³⁾ for 3 min, followed by application in the lingual/palatine at the same time. Repeat irradiations twice.

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Irradiate the vestibular surface with Ultra Blue IV System⁽⁴⁾ for 2 min, followed by lingual/palatine application at the same time. Repeat irradiations twice.

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Remove the gel in the vestibular surface and inside the pulp chamber. Then clean the surface with some gauze.

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Repeat procedures from 6 to 8 once or twice more, depending on the result obtained.

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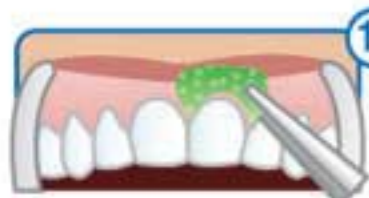
After the third application, aspirate the gel and then wash with plenty of water.

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After the end of the whitening session, make a temporary restoration by applying temporization, if necessary.

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If the desired color is not reached after the first session, repeat the entire procedure after 7 days. OBS.: If the Whitening Lase⁽³⁾ equipment is used to whiten pulpless teeth, both side sectors should be turned off.

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The results presented herein are merely illustrative

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- (1) - CREA (Regional Council of Engineering, Architecture and Agronomy)
- (2) - CRQ (Regional Council of Chemistry)

DMC Equipamentos Ltda EPP

Rua Sebastião de Moraes, 831

São Carlos - SP - Brasil - CEP / ZIP CODE 13.562-030

Tel. / Phone: (55) (16) 3362-2323

Fax: (55) (16) 3501-1830

CNPJ 02.827.605/0001-86

DMC USA Corp. (USA Branch)

12550 Biscayne Blvd #500

North Miami, Florida, USA

Zip Code 33181

Phone: 1 305 459-2097

Fax: 1 305 459-2098

www.dmc-dental.com.br